

CHEMOTHERAPY SPILL CLEAN-UP KIT (CYTOTOXIC DRUG)

This Kit is designed for the clean-up of spills involving cytotoxic drugs. The Kit contains GREEN Z fluid control solidifier to contain hazardous chemical spills. For quick & easy clean-up and disposal with minimal exposure.

HOW TO PROCEED WITH ACCIDENTAL SPILLS


The following simple steps are recommended for fast, safe clean-up.



- 1 Put on disposable gloves and appropriate protective clothing.



- 2 Sprinkle Green Z over spilled area. Liquid will quickly congeal for safer handling.

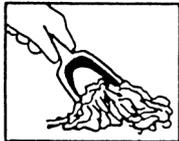

- 3 Remove gelled / solidified material with scoop and scraper. Carefully place the waste in the purple cytotoxic bag. Clean away any remaining solids with the absorbent Super Zorb Sheet.



- 4 Place all contaminated materials (including gloves) into the Purple Cytotoxic Bag. Clean hands with 'Paws' antimicrobial wipe & dispose of wipe into bag. Seal bag with twist ties and dispose of according to Regulations for infected solid waste.

CHEMOTHERAPY SPILL CLEAN-UP KIT (CYTOTOXIC DRUG)

CONTENTS:

- 1 Fluid Impervious Gown
- 1 Safety Shield
- 2 Pair Nitrile Gloves
- 1 Green Z, 142 gram bottle
- 2 Scoop & Scraper
- 1 Barrier Super Zorb Sheet (30cm x 30cm)
- 1 Cytotoxic container RE4LCT sharps disposal
- 1 Purple Cytotoxic Bag - 72 litre
- 2 Wiper Pads
- 2 'Paws' Antimicrobial Wipe
- 2 Twist Ties

DIRECTIONS FOR USE:

- ⇒ Always wear protective clothing.
- ⇒ Sprinkle Green Z solidifier over spill evenly.
- ⇒ Remove solidified material with scoop and scraper.
- ⇒ Clean area with hard surface wipe or disinfectant.
- ⇒ Remove protective clothing.
- ⇒ Clean hands with antimicrobial wipe or antiseptic cleaner.
- ⇒ Dispose of all waste in accordance with Regulations.

Reorder code:

CT42010 Premium Chemotherapy Kit

Protect Yourself - Your Staff - Your Family

idc medical ACN 078 760 504 ABN 84 078 760 504

16/13 Gibbens Road, West Gosford
NSW Australia 2250

Phone: 02 4321 0299
Internet: www.idcmedical.com

Fax: 02 4321 0288
eMail: sharps@idcmedical.com