

RE4LS – 4.5 litre Sharps Container

Assembly notes:

(to be read in conjunction with IDC Waste Disposal Safes Instructions - attached)

The RE4LS container is designed so that, when assembled, both the Lid & Fingerguard fit **TIGHT**. This tight fit ensures the integrity of the RE4LS to safely contain hazardous medical waste.

How to Fit the Lid

1. Put the Base Unit on a level surface
2. Place the lid onto the top of the container
3. Start at one corner, & with the heel of your hand press down firmly
4. When pressing down, the motion should be fast & firm to engage the Lid onto the Base.
5. Repeat the process at the next adjacent corner (not diagonally opposite), & then remaining corners.
6. The Lid is now correctly fitted onto the Base

How to Fit the Fingerguard

- The Fingerguard ‘click-seals’ into the 90mm Lid opening.
- Put the Base unit on a stable level surface
- Place the Fingerguard onto the Lid opening & use the heel of your hand to firmly press ‘click’ the Fingerguard into the opening (aperture).
- The RE4LS is now ready to use.

Locking the Container when full (or ready for disposal) – **Final Closure**

- a. The Final Closure (Red), ‘click-seals’ into the top of the Fingerguard
- b. Place the Final Closure over the top of the Fingerguard
- c. Select a section of the Final Closure at the circumference edge & the push down firmly to engage the Final Closure into the Fingerguard. Repeat process around the circumference until the Final Closure is fully sealed.
- d. The RE4LS is now locked & ready for disposal – transport to place of incineration or disposal according to local regulations.