

Operating Instructions for the use of:

Idc Rigi-Mini

for 'Sharps & Contaminated Medical Waste' Model: RIGI 0.300 ml

##Sharp and pointed objects, or contaminated articles generated during daily use, can be a danger to life and limb if used incorrectly. Please follow these Operating Instructions and Safety Guidelines.

1. Fitting

Fitting is not necessary. The Rigi-Mini is supplied fitted and fully functional.

2. Everyday use (Temporary Closure)

The Rigi-Mini is fitted with a shutter mechanism. This allows "one-handed" operation. Grip the outside walls (1) with your hand on the two longitudinal recesses (2). A forceful push with your index finger onto the opening button - "Press To Open" - (3) moves the lid cover (4) into the "open" position.

In the "open" position you can now remove or twist off suitable syringes, in respect to their cannulas. (note: points 6. pull-off / twist off device)

To close the Rigi-Mini, push the lid cover (4) down with your index finger, until the lid cover (4) springs back automatically (day seal).

The Rigi-Mini can now be transported and reopened at any time (as described above).




3. Sealing of the Day Lid (4) (FINAL Closure)

After the Rigi-Mini is FULL, proceed as follows; Place the Rigi-Mini on a firm surface and push the Lid cover (4) - the round area marked "Final Close" - firmly onto the bottom part of the Rigi-Mini (7). You will hear an audible 'click' as the lid cover (4) locks into place.

Caution: do not fill higher than the level indicator ("Fill-Line") (5) on the container.

4. Level indicator ("Fill-Line"), maximum fill height (5)

There is a rotary seam (8) located at the top section of the Rigi-Mini. The maximum fill height is positioned about 3 cm below the rotary seam (8) and, for safety reasons, must not be exceeded.

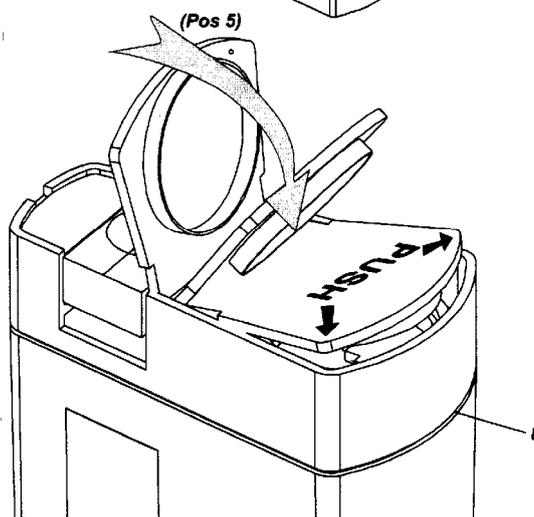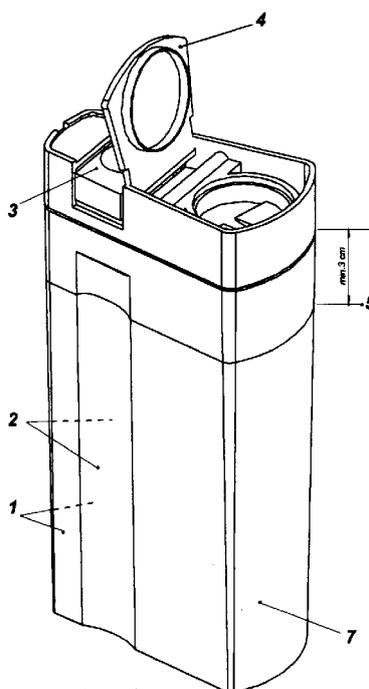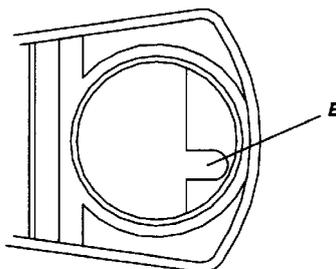

5. Brackets

A bracket or adapter is currently not intended for the Rigi-Mini.

6. Pull-off/Twist-off device

6.1 Semicircular recess (B)

This notch (B) is used to remove normal cannulas from the syringe. If the used blood collecting system fits into this recess (B), turn the cannula anti-clockwise until it falls into the bottom part of the Rigi-Box mini.

(i) Use for the Proper Purpose

This Container is only for use in the disposal of medical waste; eg. cannulas, syringes, scalpels and broken glass. Using it in any other way is contrary to the proper purpose.

If used contrary to the proper purpose, IDC Medical is not liable. Always follow the guidelines prescribed by the Infection Control Association. Dispose of properly according to regional regulations & requirements.

(ii) General Description

The Rigi-Mini is manufactured from environmentally friendly polypropylene, (bottom part of the Box), a metal spring and a polyamide lid.

There are no harmful emissions if the Rigi-Mini is incinerated.

Note:

The images and information in these Operating Instructions are subject to technical change if necessary to improve the product, & we reserve the right to do so.

Safety Guidelines

- Do not reach into the Rigi-Mini with your hand or in any other way (risk of injury)
- Do not push any objects forcefully into the Rigi-Mini
- Do not attempt to press the sides of a filled Rigi-Mini together from the outside in any way.
- Before disposing of the Rigi-Mini, make sure, that the Lid (4) is tightly sealed.
- Scalpels should be disposed of with extra care.
- If too much force is used while inserting objects or filling the Rigi-Mini with other objects, there is the danger of tilting and / or damaging the Rigi-Mini wall or the bottom of the Rigi-Mini.
- Do not pour fluids into the Rigi-Mini
- Follow the operating instructions and pay attention to the 'Biohazard Label' on the Rigi-Mini
- Do not shake the Rigi-Mini. Do not drop the Rigi-Mini
- Before closing the Rigi-Mini, make sure there are no objects protruding out. If that is the case, remove them.
- Always close the Rigi-Mini by hand, never use force
- Should the operating instructions and safety guidelines not be followed, IDC Medical cannot be held liable.
- **Sharp and pointed objects do not belong in plastic bags or other unsuitable containers.**

IDEAS DEVELOPMENT CORPORATION PTY. LIMITED

ABN 84 078 760 504 ACN 078 760 504

Incorporating IDC Medical

Phone : 02 4321 0299

Fax : 02 4321 0288

Web site : www.idcmedical.com

e-mail : sharps@idcmedical.com

OFFICE & WAREHOUSE

MAILING ADDRESS

16 / 13 Gibbens Road, West Gosford NSW

PO Box 6290, Kincumber NSW 2251