

INSTRUCTIONS FOR USE

QSSI35 | QSSI60



DANGER - NOT FOR SHARPS USE. The Steri Safe is intended for compatible clinical waste. Dispose of needles, blades and other sharps in an authorised sharps container.

1 INTENDED USE AND CORE ADVANTAGES

The Steri Safe 35 and 60 litre containers are large-format clinical-waste collection systems designed for bulky, non-sharp healthcare waste before steam sterilisation.

Material: Environmentally friendly polypropylene, guaranteed to be free of any harmful chemicals.

Form: A clean, professional appearance and square profile support efficient filling, particularly for bulky waste.

**STERI 35 / 60**

35 L: QSSI35

60 L: QSSI60

Clinical waste

Not for sharps

2 STORAGE, STACKING AND HANDLING

EMPTY STORAGE

Empty containers **inter-stack or nest** inside one another for compact, space-saving storage.

Keep clean and dry before use.

FULL STORAGE

When full, the upper container base fits into the lid vault below, fastening the units together for stable stacking.

HANDLING

Two handle hollows on each side allow one- or two-handed carrying according to weight. The centre lid handle assists opening, closing and carrying after final lock.

3 SPECIAL LID CLOSING SYSTEM

OPEN POSITION - DAILY FILLING

In OPEN, the lid cannot be sealed unintentionally. An inserted locking device prevents the cams around the base perimeter from engaging, avoiding premature disposal of a partly filled container.

CLOSE POSITION - FINAL SEAL

Turn the lid **90 degrees** into CLOSE, then push firmly downward to engage the cams in the base. The container is permanently sealed and ready for sterilisation.

4 AUTOCLAVE STERILISATION



Due to the security of the closure system the STERI is safe to autoclave. During a vacuum-steam-vacuum cycle at **110 degrees C**, the red lid cap releases and the bar-shaped steam opening allows hot vapour to enter while retaining the waste. The lid remains attached and the container retains its shape. After sterilisation, insert a new red lid cap to help neutralise latent odours. The stable, space-saving container may then be stacked and stored if required.

5 SAFETY, PRODUCT DETAILS AND DOCUMENT NOTICE



NOT FOR SHARPS. Keep the final seal engaged before transport, sterilisation, stacking or disposal.

Models: QSSI35 / QSSI60 **Type:** Autoclavable clinical-waste container
Document: IFU-QSSI35-QSSI60 **Revision:** 2026.07
Support: sales@idcmedical.com

DISCLAIMER: IDC Medical do not warrant or represent in any way whatsoever that IDC products (Products) meet current standards (including AS23907-2:2019). You expressly acknowledge and agree that you are responsible for ensuring that your use of the Products are compliant for your purposes. You expressly indemnify IDC Medical from and against any claims, actions and all other liability with respect to your use of the Products.

V 202605 - The images and / or information provided in this document are for general guidance only and may be subject to technical changes. We reserve the right to modify product specifications, design, or information where necessary. Images are for illustrative purposes only and may differ from the actual product.