

INSTRUCTIONS FOR USE

QS 0.5



WARNING: Used sharps can cause serious injury and infection. Read these instructions before use. Only trained or appropriately instructed users should handle the container.

1 PURPOSE, MATERIAL AND TRACEABILITY

INTENDED USE

The Quick Safe 500 mL is intended only for disposal of authorised medical sharps waste, including cannulas, syringes, scalpels and broken glass. Any other use is contrary to the intended purpose. IDC Medical is not liable for use contrary to that purpose. Follow applicable infection-control guidance and regional disposal requirements.

GENERAL DESCRIPTION

Manufactured from environmentally friendly polypropylene. There are no harmful emissions if the container is incinerated. The production date (month/year) and serial number are stated on the container for batch tracing purposes.

2 FIT, OPEN AND CLOSE THE LID

1. FITTING THE LID


Before filling, place the container on a firm surface. Press the Lid firmly onto the Base until the lid locks into place with an audible engagement.

2. EVERYDAY USE - TEMPORARY CLOSURE

For temporary closure during everyday use, lightly press the Day Lid into the Lid Opening. Reopen the day lid when the container is required.

3. PERMANENT FINAL CLOSURE

When the container is full, place it on a firm surface and press the Day Lid firmly into the Lid Opening until it locks with an audible "click". Do not fill above the maximum fill line. Final closure is permanent.

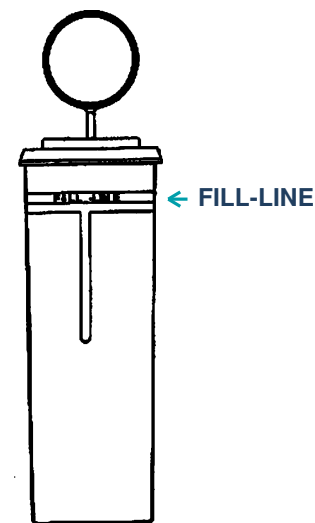

Quick Safe 500 mL

3 MAXIMUM FILL HEIGHT AND PRODUCT IDENTIFICATION

4. MAXIMUM FILL HEIGHT

The level indicator - Fill Line - is visible at the top of the container. Do not fill above this line under any circumstances. Exceeding the maximum fill height compromises safe closure and handling.

Before final closure:


Confirm that no item projects above the fill line or obstructs the lid opening.

INSTRUCTIONS FOR USE

QS 0.5

4 UNIVERSAL BRACKET WITH ADHESIVE - QSLOCK

- **Fit:** Turn the container over. Insert the Universal Bracket (6) into the adaptor recess in the base and turn clockwise.
- **5.1 Position:** Determine where you want to position the Container and in which direction the pull-off / twist-off device will point. This position must be cleaned of any grease and dust.
- **5.2 Bond:** Remove the protective film, position the container and bracket, then press firmly. Twist off the container and again press firmly around the bracket edge to confirm a secure bond.
- **5.3 Reuse:** A new container may be attached to the bracket as often as required. The installed bracket stabilises the container. Screw holes are provided where mechanical fixing is preferred.
- **5.4 Container to bracket:** Turn clockwise to lock the container onto the bracket. After the final closure, turn anticlockwise to remove the container.

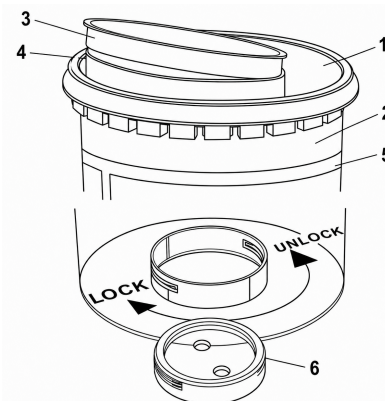

5 PULL-OFF AND TWIST-OFF DEVICES

6.1 JAGGED NOTCH (A)


For blood-collection systems, insert the cannula lug until the edges interlock. Turn anticlockwise until the cannula twists off and falls into the base.

6.2 SEMICIRCULAR RECESS (B)

Use for normal cannulas. If the blood-collection system fits the recess, turn anticlockwise until the cannula falls into the base.

6.3 MULTI-TOOTHED OPENING (C)

Position the insulin-pen needle in the spoked opening and rotate the pen anticlockwise. Push the detached needle slightly deeper so it falls into the container.

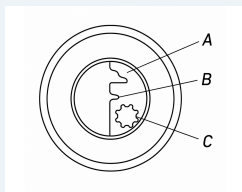

6 SAFETY GUIDELINES

- Before filling, confirm the Lid is fitted and locked.
- Do not reach into the container by hand or in any other way.
- Do not push objects forcefully into the container.
- Do not press the sides of a filled container together.
- Before disposal, confirm the Lid is tightly closed into the container Lid Opening.
- Handle scalpels with additional care.
- Excessive insertion force may tip the container or damage its wall or base.
- Do not pour fluids into the container.
- Secure the container with the recommended QSLOCK bracket to prevent tipping.

- Follow these instructions and the Biohazard Label.
- Do not shake or drop the container.
- If an item protrudes, do not attempt closure or reach inside. Remove it only through your facility's approved sharps-handling procedure.
- Close the container by hand only; never use force.
- Do not use the hinged lid to push waste into the container.
- Deposit hazardous material vertically, never transversely.
- After final closure, do not press the container against your body.
- IDC Medical cannot be held liable if these instructions are not followed.
- Sharps do not belong in plastic bags or other unsuitable containers.

Disclaimer: IDC Medical do not warrant or represent in any way whatsoever that IDC products (Products) meet current standards (including AS23907-2:2019). You expressly acknowledge and agree that you are responsible for ensuring that your use of the Products are compliant for your purposes. You expressly indemnify IDC Medical from and against any claims, actions and all other liability with respect to your use of the Products.

V 202605 - The images and /or information provided in this document are for general guidance only and may be subject to technical changes. We reserve the right to modify product specifications, design, or information where necessary. Images are for illustrative purposes only and may differ from the actual product.