

INSTRUCTIONS FOR USE

QSeu 4.0 & 7.0



WARNING: Used sharps can cause serious injury and infection. Read these instructions before use. Only trained or appropriately instructed users should handle the container.

1 PURPOSE, MATERIAL AND TRACEABILITY

INTENDED USE

This Container is only for use in the disposal of medical waste; e.g. cannulas, syringes, scalpels and broken glass. Using it in any other way is contrary to the proper purpose, IDC Medical is not liable. Always follow the guidelines prescribed by the Infection Control Association. Dispose of properly according to regional regulations & requirements.

GENERAL DESCRIPTION

Manufactured from environmentally friendly polypropylene. There are no harmful emissions if the container is incinerated. The production date (month/year) and serial number are stated on the container for batch tracing purposes.

2 FIT, OPEN AND CLOSE THE LID

1. FITTING THE LID


Before filling, place the container on a firm surface. Press the top part - Lid (1) firmly onto the bottom part - Base Unit (2) until the lid locks into place.

2. EVERYDAY USE - TEMPORARY CLOSURE

The container opens automatically at the touch of the button (Pic 1). For temporary closure, lightly press the Day Lid (3) into the Lid Opening (4) (Pic 2). Reopen the day lid when the container is required.

3. PERMANENT FINAL CLOSURE

When the container is full, place it on a firm surface and press the Day Lid (3) firmly into the Lid Opening (4) until it locks with an audible "click". The final carrying handle is released automatically (Pic 4).

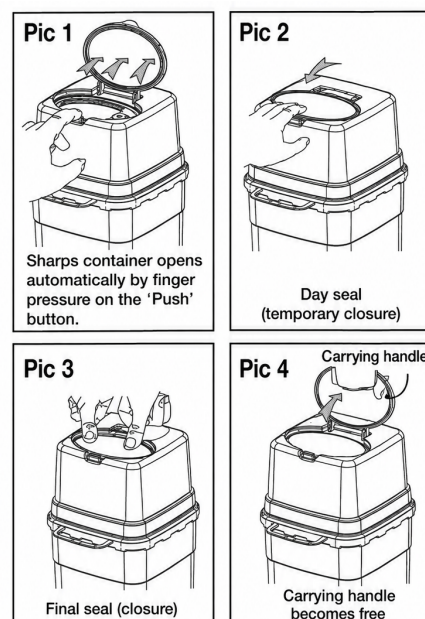


3 FILL LINE, CARRYING HANDLES AND PRODUCT COMPONENTS

4. MAXIMUM FILL HEIGHT

The level indicator (5) is visible at the top section of the container. Do not fill above this line under any circumstances. The maximum fill height must not be exceeded for safety reasons.

6. CARRYING HANDLES

Before final closure, carry the container using holding device (9). When final closure engages, carrying handle (10) springs up automatically (Pic 4). This carrying handle signifies that the Container is now finally sealed and can easily be transported to the place of disposal.

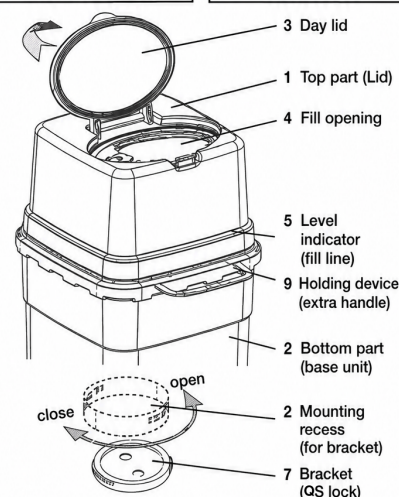


INSTRUCTIONS FOR USE

QSeu 4.0 & 7.0

4 UNIVERSAL BRACKET WITH ADHESIVE - QSLOCK

- **Fit:** Turn the container over. Insert the Universal Bracket (7) into the adapter recess (8) in the base and turn clockwise.
- **5.1 Position:** Determine where you want to position the Container and in which direction the pull-off/twist-off device will point. Clear the area of any grease and dust.
- **5.2 Bond:** Remove the protective film, position the container and bracket, then press firmly. Twist off the container and press firmly around the bracket edge to confirm a secure bond.
- **5.3 Reuse:** A new container may be attached to the bracket as often as required.
- **5.4 Alternative fixing:** Screw holes are provided where mechanical fixing is preferred to the adhesive method.
- **5.5 Container to bracket:** Turn clockwise to lock the container onto the bracket. Turn anticlockwise to remove the container.

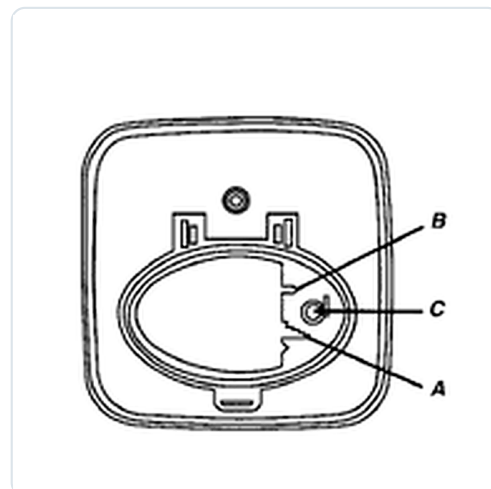

5 PULL-OFF, TWIST-OFF AND SCALPEL-DETACHMENT DEVICES

7.1 JAGGED NOTCH (A)

For blood-collecting systems, insert the cannula lug into the indented recess until the edges interlock. Turn anticlockwise until the cannula twists off and falls into the base.

7.2 SEMICIRCULAR NOTCH (B)

Use for normal cannulas. If the blood-collecting system fits the recess, turn anticlockwise until the cannula falls into the base.

7.3 SCALPEL DEVICE (C)

Insert the scalpel into the opening until it snaps in and the blade lifts slightly. Pull out the scalpel shaft; the blade falls into the sharps container.

6 SAFETY GUIDELINES

- Before filling, confirm Lid (1) is fitted and locked.
- Do not reach into the container by hand or in any other way.
- Do not push objects forcefully into the container.
- Do not press the sides of a filled container together.
- Before disposal, confirm the Day Lid is closed tightly into the Container Lid Opening
- Handle scalpels with additional care.
- Excessive insertion force may tip the container or damage its wall or base.
- Do not pour fluids into the container.
- Secure the container with the recommended QSLOCK bracket to prevent tipping.

- Follow these instructions and the Biohazard Label.
- Do not shake or drop the container.
- If an item protrudes, do not attempt closure or reach inside; follow your facility procedure.
- Close the container by hand only; never use force.
- Do not use the hinged lid to push waste into the container.
- Deposit hazardous material vertically, never transversely.
- After final closure, do not press the container against your body.
- IDC Medical cannot be held liable if these instructions are not followed.
- Sharps do not belong in plastic bags or other unsuitable containers.

Disclaimer: IDC Medical do not warrant or represent in any way whatsoever that IDC products (Products) meet current standards (including AS23907-2:2019). You expressly acknowledge and agree that you are responsible for ensuring that your use of the Products are compliant for your purposes. You expressly indemnify IDC Medical from and against any claims, actions and all other liability with respect to your use of the Products.

V 202605 - The images and /or information provided in this document are for general guidance only and may be subject to technical changes. We reserve the right to modify product specifications, design, or information where necessary. Images are for illustrative purposes only and may differ from the actual product.