

QS Series Sharps Container - SANI-Safe 200 ml Cannula

INSTRUCTIONS FOR USE

QSsnC



WARNING: Used sharps can cause serious injury and infection. Read these instructions before use. Only trained or appropriately instructed users should handle the container.

1 PURPOSE AND PRODUCT OVERVIEW

The IDC Medical SANI-Safe 200 ml is a **single-use sharps container** designed primarily for detached cannulas and other compatible small sharps generated during healthcare procedures.


No fitting is required. The container is supplied fully assembled, fitted and functional.

Suitable items include:

- Detached cannulas and compatible needle components
- Other small sharps that pass freely through the opening
- Small broken-glass fragments where facility procedure permits

Model-specific restriction: Do not deposit complete syringes or scalpels in the SANI-Safe 200 ml. Use an appropriately sized authorised sharps container.

PRODUCT FEATURES

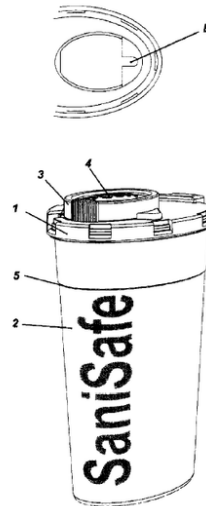


- 2 Container body
- 3 Turning lid
- 4 Permanent-lock PUSH panel
- 5 Maximum fill indicator
- B Semicircular twist-off recess

2 BEFORE FIRST USE

- ✓ **Inspect the container.** Do not use if cracked, distorted, contaminated, previously permanently closed or otherwise damaged.
- ✓ **Confirm the lid operates freely** between OPEN and CLOSE and the PUSH panel is raised and undamaged.
- ✓ **Select a safe location** close to the point of use and away from children, patients and public access.
- ✓ **Keep the container upright and stable.** Ensure the fill indicator remains visible.

3 EVERYDAY USE, TEMPORARY CLOSURE AND TRANSPORT

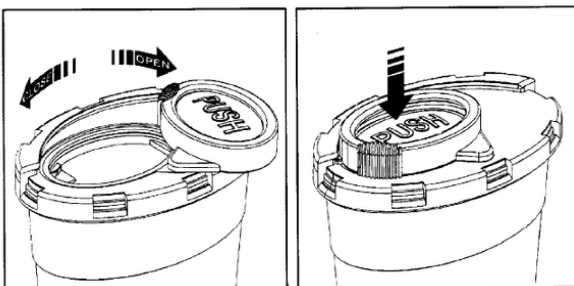

OPEN

Turn the lid with one hand to expose the deposit opening. Keep the container upright and deposit compatible items vertically.

TEMPORARY CLOSE

Turn the lid to CLOSE for safe transport. The container may be reopened until the permanent PUSH closure has been activated.

MAXIMUM FILL INDICATOR

Monitor the indicator at the top of the container. Never fill above it; close earlier if contents obstruct the lid.

SEMICIRCULAR TWIST-OFF RECESS (B)

If a compatible cannula or blood-collection hub fits the recess, rotate the syringe anti-clockwise until the cannula falls in. If it does not release easily, stop.

QS Series Sharps Container - SANI-Safe 200 ml Cannula

INSTRUCTIONS FOR USE

QSsnC

4 PERMANENT FINAL CLOSURE

Activate only when the container is ready for disposal. Permanent closure cannot be reversed. Keep hands clear of the deposit opening.

1

CHECK FILL LEVEL

Place the container on a firm, level surface. Confirm no item protrudes and the fill indicator has not been exceeded.

2

TURN TO CLOSE

Using one hand, rotate the turning lid (3) fully into the CLOSE position.

3

PRESS PUSH

Press the surface marked PUSH (4) firmly by hand into the base section of the container.

4

VERIFY

Confirm the lid cover locks audibly into place and cannot be reopened.

✓ **Final check:** The PUSH panel must be firmly engaged before disposal. Do not use it to push waste into the container and never force the lid.

5 SAFETY, HANDLING AND DISPOSAL

ALWAYS

- ✓ Follow the operating instructions, biohazard label, infection-control requirements and regional clinical-waste regulations.
- ✓ Deposit compatible sharps vertically and allow them to fall freely.
- ✓ Exercise extra care with blades and broken glass; use a larger authorised container when required.
- ✓ Close the lid by hand and keep the container upright during handling and temporary transport.
- ✓ Before permanent closure, stop use if an item protrudes. Do not reach in; follow the facility's safe obstruction procedure.
- ✓ Transfer the permanently sealed container to an authorised clinical-waste service.

DO NOT

- Reach into the container or attempt to retrieve an item.
- Force objects into the opening, compact contents or press the sides together; this may tip or damage the container.
- Pour free liquids into the container.
- Shake or drop the container.
- Use the PUSH panel to force waste into the container.
- Deposit complete syringes or scalpels in this 200 ml model.
- Transport after permanent closure unless the lock is fully engaged.
- Place sharps in plastic bags or other unsuitable containers.

6 PRODUCT DETAILS, MATERIAL AND SUPPORT

INTENDED PURPOSE AND DISPOSAL

Use only for authorised medical sharps waste. General sharps categories include cannulas, syringes, scalpels and broken glass; however, complete syringes and scalpels are prohibited in this 200ml model. Should operating instructions and safety guidelines not be followed, IDC Medical cannot be held liable. Dispose of the container through an authorised service.

PRODUCT AND DOCUMENT DETAILS

Model	QSsnC
Product	SANI-Safe 200 ml
Document	IFU-QSsnC
Revision	2026.07
Support	sales@idcmedical.com

DISCLAIMER

IDC Medical do not warrant or represent in any way whatsoever that IDC products (Products) meet current standards (including AS23907-2:2019). You expressly acknowledge and agree that you are responsible for ensuring that your use of the Products are compliant for your purposes. You expressly indemnify IDC Medical from and against any claims, actions and all other liability with respect to your use of the Products.

V 202605 - The images and/or information provided in this document are for general guidance only and may be subject to technical changes. We reserve the right to modify product specifications, design, or information where necessary. Images are for illustrative purposes only and may differ from the actual product.