

RE Series Cytotoxic Containers

INSTRUCTIONS FOR USE

RE4LCT | RE4LCTR
 RE10LCT | RE20LCTR

EACH CYTOTOXIC SAFE (E) IS SUPPLIED WITH

A Final Closure B Deposit-Guard C Lid D Locking Ring E Cytotoxic Safe / Base Unit

1 ASSEMBLY AND SET-UP


1 FIT AND LOCK THE DEPOSIT-GUARD

Fit the **Deposit-Guard (B)** into the opening in the **Lid (C)**. Screw the **Locking Ring (D)** onto the thread of the Deposit-Guard. Confirm that the Deposit-Guard is securely locked to the Lid.

2 ATTACH THE LID TO THE BASE UNIT

Position the **Lid (C)** on the **Cytotoxic Safe / Base Unit (E)**. Starting at one point, press firmly downward, then work around the full circumference in one direction until the Lid is completely secure.

PRODUCT COMPONENTS

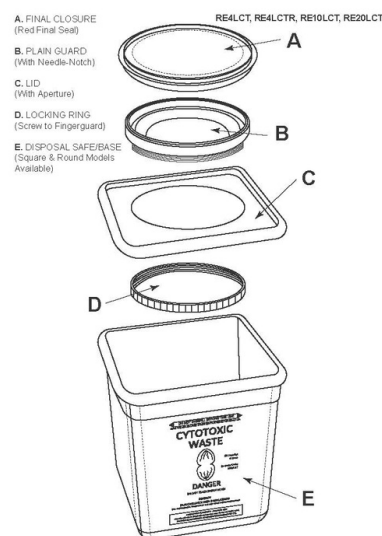

2 PERMANENT FINAL CLOSURE

Seal only when ready for disposal

When the Cytotoxic Safe is full or ready for disposal, place the **Final Closure (A)** directly into the **Deposit-Guard (B)** aperture and press until it clicks securely into place. The sealed container is then ready for disposal in accordance with regional regulations and requirements.

PERMANENT SEAL - DO NOT REOPEN. Once the Final Closure is sealed into the Deposit-Guard, the Cytotoxic Container cannot be reopened. Always install and use the Deposit-Guard when the Cytotoxic Safe is used for cytotoxic-waste disposal.

3 SAFE USE REMINDERS

IMPORTANT / ALWAYS

- ✓ Polymer products, although puncture resistant, are **not entirely puncture proof**.
- ✓ Follow the guidelines set down by the Infection Control Association.

NEVER

- ✓ Fill above the indicator fill line.
- ✓ Force an article into the Cytotoxic Safe.
- ✓ Deposit a syringe with the needle pointing upwards.

Disclaimer: IDC Medical do not warrant or represent in any way whatsoever that IDC products (Products) meet current standards (including AS23907-2:2019). You expressly acknowledge and agree that you are responsible for ensuring that your use of the Products are compliant for your purposes. You expressly indemnify IDC Medical from and against any claims, actions and all other liability with respect to your use of the Products.
 V 202605 - The images and/or information provided in this document are for general guidance only and may be subject to technical changes. We reserve the right to modify product specifications, design, or information where necessary. Images are for illustrative purposes only and may differ from the actual product.