

INSTRUCTIONS FOR USE

ZEO-BZ001
ZEO-BZ002 | REFILL



BIOHAZARD SPILL: Avoid cross-contamination. Do not touch the inside of the gusseted clinical-waste bag.

1 PREPARE THE RESPONSE

1 DON PROTECTIVE CLOTHING

Empty the kit contents onto a clean surface. Don the protective clothing: **apron, gloves and face mask with wrap-around eye shield.**

2 PREPARE THE GUSSETED BAG

Open the gusseted clinical waste bag (yellow) by placing hands inside the bag and firmly pressing on the base. Set it upright in a stable, free-standing position.

2 ABSORB AND CONTAIN THE SPILL

Maintain contamination control

3 APPLY ABSORBENT POWDER

Open the ZeoMed absorbent powder **sachet**. Apply the powder around the outside perimeter of the spill first to contain the fluid. Then sprinkle generously over the remainder of the spill. Allow **1-2 minutes** for the powder to absorb the spill and odours.

4 COLLECT THE ABSORBED MATERIAL

Using the scraper and scoop, collect the substance and place it into the flat standard bio-waste bag (yellow). Place the scraper and scoop into the same bag. **Avoid cross-contamination. Do not touch the inside of the gusseted bag.**

3 CLEAN, SEAL AND COMPLETE

5 CLEAN AND DRY THE AFFECTED AREA

Clean the area with the **Clinical Detergent Solution (use all of the solution)** and one supplied cleaning cloth. Contain the spill by wiping from the outside towards the centre. Place the contaminated cloth into the gusseted clinical waste bag (yellow). Dry the area with the second cloth and place it in the same bag. **Avoid cross-contamination. Do not touch the inside of the gusseted bag.**

6 REMOVE PPE, SEAL AND DISPOSE

Remove the apron, face mask and gloves and place them in the gusseted clinical bag (yellow). Seal securely with **both ties**, then dispose of the bag in accordance with Government regulations.

7 COMPLETE HAND HYGIENE

Wash hands thoroughly with **soap and water**.



Completion check: Confirm the spill area is clean and dry, contaminated items and PPE are sealed in the yellow bag, disposal requirements have been followed and hand hygiene is complete.

V 202605 - The images and/or information provided in this document are for general guidance only and may be subject to technical changes. We reserve the right to modify product specifications, design, or information where necessary. Images are for illustrative purposes only and may differ from the actual product.