

ZeoMed: Spill Absorbent Powder

INSTRUCTIONS FOR USE

ZEO-Z006 | ZEO-Z00501
ZEO-Z00505 | ZEO-Z00510



WARNING: May cause respiratory irritation. Avoid breathing dust. Use only outdoors or in a well-ventilated area.

1 PURPOSE AND AVAILABLE FORMATS

INTENDED USE

ZeoMed Spill Absorbent Powder is supplied for absorbing a **body-fluid spill** or **associated odour** before the material is collected, contained and disposed of.

REORDER CODES

ZEO-Z006	200 g sachet
ZEO-Z00501	1 kg tub
ZEO-Z00505	5 kg tub
ZEO-Z00510	10 kg tub

2 SAFE APPLICATION AND DISPOSAL

Complete each step in sequence

1 OPEN AND APPLY

Open the selected **sachet, pouch or tub**. Apply powder around the outside perimeter of the spill first to contain the fluid, then sprinkle generously over the remainder of the spill.

2 ALLOW TO ABSORB

Wait **1-2 minutes**, or until the powder has absorbed the spill or odour.

3 COLLECT AND CONTAIN

Using a **scoop and scraper**, collect the absorbed material. Place it in a biohazard bag and seal the bag securely.

4 DISPOSE

Dispose of infectious waste in compliance with applicable **Local Government regulations**.

3 SAFETY INFORMATION AND SDS



RESPIRATORY IRRITATION

May cause respiratory irritation. Avoid breathing dust. Use only outdoors or in a well-ventilated area.
IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Call a **POISON CENTRE**, doctor or physician if you feel unwell.
Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to an approved waste disposal plant.



SCAN HERE FOR SDS

V 202605 - The images and/or information provided in this document are for general guidance only and may be subject to technical changes. We reserve the right to modify product specifications, design, or information where necessary. Images are for illustrative purposes only and may differ from the actual product.