

INSTRUCTIONS FOR USE

ZEO-ZCYTB

- 1** Do not leave the spill unattended. Take care not to contaminate yourself with any spill material.
- 2** Call for assistance and locate the spill kit.
- 3** Place cytotoxic spill signage at the door and near the spill.
- 4** Put on the protective clothing in the following order:
 - 1 - Mask with eye shield
 - 2 - Hairnet
 - 3 - Gloves - both pairs
 - 4 - Protective gown
 - 5 - Shoe covers
- 5** Empty the remaining contents of the kit onto a nearby clean surface.
- 6** Remove any contaminated clothing and place it in the red alginate soluble laundry bag. Use the tie to secure the bag and place it in the contaminated laundry bag. If skin has come into contact with body fluid, rinse well with water.
- 7** Gently pour the absorbent powder onto the spill to avoid further spread. **Please note:** Sprinkle the powder beginning at the outer edge, then work towards the middle until the entire spill is covered. Wait a few minutes.
- 8** Carefully place one of the absorbent cloths over the spill. This cloth may also be used to remove the bulk of the spill.
- 9** Open one purple cytotoxic bag. Using the scoop and scraper provided, scrape up the congealed waste and place it into the cytotoxic bag. Immediately place the scoop and scraper into the same bag as the waste.
- 10** Pour the detergent provided over the spill area and clean with the second cloth. Place the used cloth into the same purple bag.
- 11** Remove your shoe covers, followed by your outer gloves. Carefully place these items into the same purple bag.
- 12** Tie off the bag with a cable tie provided, then place it into the second purple bag.
- 13** Remove your face mask with eye shield and place it in the purple bag. Do the same with your protective gown, followed last of all by your second pair of protective gloves. Avoid contact with any residue on the bag.
- 14** Tie off the second bag containing all items.
- 15** Dispose of the waste in accordance with Hospital Policy. Complete an incident report form provided by your organisation and inform your management.
Order Code: ZEO-ZCYTB