

INSTRUCTIONS FOR USE

ZEO-ZCYTQ

1 Alert people in the immediate vicinity that a Cytotoxic Chemical Spill has occurred & instruct them to remain clear. Remain with spill, taking care not to walk in, stand in or create a draft around spill.

2 Ask person involved to remove contaminated clothing quickly and have them wash their affected skin thoroughly with soap and water.

3 Open the Cytotoxic Chemical spill kit, remove Cytotoxic spill signs and display around perimeter of the spill. Limit access to spill, call for assistance.

4 Empty contents of kit within reach of spill.

5 Contain spill (if liquid) by placing absorbent cloths around perimeter.

6 Remove any contaminated clothing and place in the red alginate soluble laundry bag. Use tie to secure bag and place in contaminated laundry bag. If skin has come in contact with body fluid rinse well with water

7 Don Personal Protection Equipment in this order:
1 - Mask
2 - Eyewear
3 - Hairnet
4 - Chemo gloves
5 - Chemo gown (ensure cuffs of gown are over the 1st pair of gloves)
6 - Overshoes
7 - Followed by 2nd pair of chemo gloves (gloves are to go over top of gown cuffs).

8 Open two (2) Cytotoxic waste bags, placing one inside the other (slightly roll tops of bags down to reduce contamination).

9 For spills - (take care not to generate aerosols)
1 - Liquid spills - carefully cover spill with Chemosorb Pad
2 - Powder spills - place Chemosorb pad over the powder, saturate pad with water (without flooding) until powder dissolves & is absorbed. Cytotoxic powder should now be absorbed by the now gel coagulant).

10 Using the scoop and scraper, scoop up waste matter and dispose of into inner Cytotoxic waste bag.

11 Moisten absorbent cloths with clinical detergent and commence cleaning from the outer aspect of the spill working inwards. Place used cloths into the Cytotoxic waste bag. Continue using cloths until spill area is completely cleaned. **At least (3) washes.**

12 Rinse spill area thoroughly with water.

13 Dry area with any remaining cloths and dispose of all items into inner waste bag

14 Remove outer gloves and dispose of into inner waste bag. Seal bag with cable tie.

15 Remove all additional PPE, placing into the outer waste bag. To be removed in the following order:
1 - Overshoes
2 - Chemo gown
3 - Protective eyewear
4 - Hair net
5 - Respirator and
6 - 2nd pair of gloves
7 - Tie off bag with cable tie

16 Place Cytotoxic waste bag into Rigid Cytotoxic waste bin.

17 Wash hands thoroughly with soap & water

18 Complete appropriate incident report form

19 Reorder Cytotoxic Spill Kit from ZeoMed.

Order Code: ZEO-ZCYTQ